

Baffin TRIO

Advanced set up guide

1. Before transferring the user in, please get the pelvic belts ready laying flat on the seat.



2. Place the user in the seating position (slight tilt may be used). Ensure that the back is in the right position, e.g. no gaps between the backrest and the user's back.
3. Fasten the pelvic belts to the black strap.



4. Fasten the vest to the coloured (usually the same colour as the equipments upholstery) strap. (Top straps are coming from the top of the backrest, bottom corner straps are coming from under the seat and should go round the hip)



5. Put the knee blocks on and lock them into place.

A. When putting the knee block in make sure you've got an angle to get the metal knob locked in the right place.



B. Push the knee block towards the user. When it's in the right place lock it with a handle/screw on the side.



6. **Fasten the straps on the foot platform.**
7. **Fasten the padded abduction belts if needed.**
8. **The user is secure and safe, so you can use the equipment now.**



